



Solutions to improve the quality of life

**This product may be purchased from Connevens Limited secure online store
at www.DeafEquipment.co.uk**



DeafEquipment.co.uk

Solutions to improve the quality of life



40ILRKIT – Loop Receiver Tester

Kit consists of: a plastic pouch containing a Loop Listener, a pair of headphones and an alkaline PP3 battery.

Battery: Insert battery in base.

To use: Turn on the tester by plugging in the headphones, the power 'ON' LED will light up. Turn off by unplugging headphones, LED will go out. Adjust volume to a comfortable level with the volume level knob. Clip unit to pocket or belt. Keep unit vertical in use.

LO CUT switch: To improve speech intelligibility or to reduce background noise, set the 'LO CUT' switch to ON (this will suppress low frequencies).



TO USE AS A LOOP LISTENER:

Non hearing aid wearers with a mild hearing loss can use this kit as an amplified loop listener.

Simply turn on and listen to the loop output through the headphones. Adjust volume as required.

TO TEST LOOP INSTALLATIONS:

The induction loop system needs to be turned on with a normal sound input so you will need a colleague to talk into the microphone. Walk around the looped area, ensuring that you can hear the loop signal clearly and without interference.

If you encounter any significant drop in volume (field strength) or intrusive unwanted magnetic interference (eg. lighting transformer, faulty mains wiring, data cabling etc.), try to identify the cause of the problem and rectify it.

If the loop wire installation cannot be altered or if the source of interference cannot be moved or switched off, ensure that users of the system are directed away from these areas.



CONNEVANS LIMITED

54 Albert Road North, Reigate, Surrey RH2 9YR
Information website: www.connevens.com

Helpline: 01 737 247571 help@connevens.com
Shop online: www.DeafEquipment.co.uk