

Music-Link stereo inductive silhouette earhooks

Inductive silhouette earhooks with 3.5mm stereo jack plug for use with post aural hearing aids.




The earhook provides an inductive output for use with a hearing aid 'T' input; because there is one for each ear a stereo signal can be heard by the hearing aid user.

The Music-Link can be used with any equipment with a suitable headphone socket eg. personal radios, minidisks, CD players, cassettes, talking book machines etc.

Place the earhooks over the ear between the aid and head, plug into the headphone socket and switch your hearing aid to the 'T' setting.

The silhouette earhooks are position sensitive - you need to experiment by moving them to get the best signal.

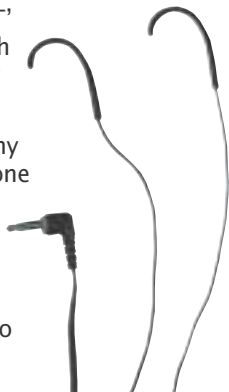

Music-Link stereo inductive silhouette earhooks
(Pt No. 95MUSICL)

*Can also be used with 3 metre stereo extension lead
(Pt No. ★A121A★)*



CONNEVANS LIMITED

54 Albert Road North, Reigate, Surrey RH2 9YR

Helpline 01737 247571

help@connevens.com