


Supplementary T-Link Instructions

The following points should be remembered when using your T-Link.

1. Experiment by moving the earhook relative to the hearing aid for the loudest/best sound.
2. Keep the mobile phone as far away as necessary from the hearing aid to avoid interference.
3. To improve outgoing voice quality hold the microphone near to the mouth during phone calls.

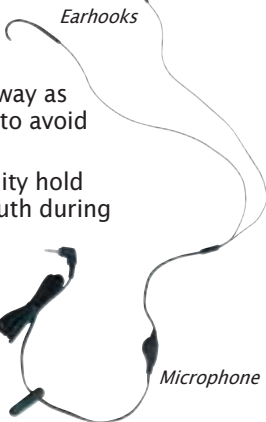

A clip is included in the pack - to be honest, most people don't use it, however it can be used to stop the lead flapping about!

The T-Link has a standard 2.5mm 3-pole jack plug which is the same as on most hands-free kits. If the mobile phone that you have requested the T-Link to work with requires an adaptor, this will be included in the pack.

If the adaptor becomes damaged or you change your mobile phone, simply purchase the appropriate adaptor from your mobile phone shop.



CONNEVANS LIMITED

54 Albert Road North, Reigate, Surrey RH2 9YR

Helpline 01737 247571

help@connevens.com

T-Link Frequently asked Questions

How does the T-Link connect to the Mobile? It connects to the hands free connector just like a standard hands free kit. Have a look at the Mobile phone manual to see where it connects, using adaptor supplied if appropriate.

Does the T-Link need any batteries? No – it is completely passive and uses the audio system of the Mobile

How will I know when the Mobile is ringing? A few of the Mobiles do put ring tones into accessory output. However if you are using the hearing aid normally, you will hear the ring. Some Mobiles do have a vibrate alert facility – ask you Mobile phone dealer.

Which types of hearing aid will the T-Link work well with? Any type of BTE (behind the ear), ITE (in the ear) or CIC (completely in canal) – it must be fitted with a T-Coil.

What if my hearing aid does not have a T switch? Take it back to your hearing aid provider. They will probably be able to modify it.

Should I try to speak into the Mobile? No – keep the Mobile as far away from you hearing aid as possible – that is why you have to have the T-Link.

I like the T-Link, is there a similar product I can use with my personal stereo and PC? There are two other products in the range called the MUSIC-Link for use with single or double hearing aid users. These work in a similar way to the T-Link.

How long are the products guaranteed for? Against faulty workmanship – for one year. However the nature of this type of product can expose it to severe physical extremes. Damage caused by sweat, rough handling and pulling is not covered. If the product stops working with no obvious damage within one year we will replace it.