

fmGenie personal Soundfield case and amplified speakers

Contents supplied: case, pair of speakers, 4 x AA batteries, velcro patch.

Also required but not supplied: fmGenie receiver and belt pouch.



Soundfield case and speakers

Fit 2 AA batteries into the speaker


pod with the On/Off switch and

place both speakers into the case with the switch by the hole so that it can be accessed from outside the case.

Use the self adhesive velcro patch to stick the two speakers together if wished. This stops the speakers rattling around in the case.



fmGenie receiver

Make sure the receiver is set to 'HI' and showing the  symbol on the display.

Initially set the tone controls to 'CANCEL' although these can be adjusted later to suit user.

Place receiver in belt pouch and attach to fixing strap on the end of the case.




Plug speakers into the 'Audio out' socket on the fmGenie receiver.

Turn on fmGenie receiver and speakers.

fmGenie transmitter

We recommend the use of an ear or headworn microphone with the fmGenie transmitter as a lapel microphone can cause feedback particularly if used too close to the speakers.



 Full information on settings can be found on page 11 of the fmGenie Fast Track leaflet or pages 72-73 of the fmGenie Information Booklet.



CONNEVANS LIMITED

Helpline 01737 247571

sales@connevens.com

54 Albert Road North, Reigate, Surrey RH2 9YR

ALL PRICES ARE SHOWN EXCLUSIVE OF SHIPPING & HANDLING AND VAT