

**This product may be purchased from Connevens Limited secure online store  
at [www.DeafEquipment.co.uk](http://www.DeafEquipment.co.uk)**



**DeafEquipment.co.uk**

Solutions to improve the quality of life



Wireless Systems

# ZoomLink

Tune in for a Crisp Sound



**PHONAK**  
hearing systems



ZoomLink by Phonak is a new transmitter with a built-in microphone and an easy, three button interface. ZoomLink features an Omni, Zoom and SuperZoom<sup>2</sup> mode, which provides the user in each listening situation with the right reception angle. ZoomLink is compatible with all Phonak receivers.

Wireless hearing systems are by far the best technology to **improve speech understanding in noise**, addressing the number one desire of adult hearing instrument users.

The MyLink receiver with the ZoomLink transmitter is a very attractive combination. MyLink is compatible with all hearing instruments with a T-coil, including custom products. Together with the easy and flexible ZoomLink transmitter, your clients will be able to tune in for a crisp sound!



## Proven Technology

### A) Hidden antenna

The antenna is cleverly hidden in the lavalier cord, which can be used to hang ZoomLink around the neck of a speaker.

### B) Easy and precise operation

ZoomLink has three easily accessible buttons. Each button functions as an on-off button. ZoomLink will start operating in Omni-directional mode, Zoom mode or SuperZoom<sup>2</sup> mode, depending on which button was pressed. A short press on any of the buttons will change the microphone mode. A long press on any of the three buttons will switch ZoomLink off again.

### C) Digital beamforming microphones

The high quality microphones in ZoomLink can operate in three different modes. In Omni-directional mode, sound is picked up equally from all directions. In Zoom mode, the beamformer has a fixed cardioid characteristic. In SuperZoom<sup>2</sup> mode, the beamformer is adaptive in 20 independent frequency channels. On top of that, a multi-frequency fast noise suppression algorithm is active in SuperZoom<sup>2</sup> mode, suppressing any residual stationary noises coming from the front. The microphones are shielded by a very effective wind and weather protection.

A

B

C



# Efficient Counselling

ZoomLink and MyLink form an ideal and affordable system for better communication in noisy situations or over distance.

## Simply show your client how to:

- switch on ZoomLink
- switch on MyLink and set the hearing instruments to MT or T
- play with the different microphone modes of ZoomLink to discover the optimal reception angle for a particular listening situation

The ZoomLink-MyLink bundle is the best you can get for better hearing in noise, when both affordability and flexibility are on your wish list.

ZoomLink is a member of the wide range of Phonak wireless transmitters. ZoomLink is compatible with all Phonak receivers, including MicroMLxS, MLxS, ML9S, iLink, MicroLink Freedom, MicroLink CIS, MicroLink Baha, EduLink and MyLink.



MyLink is a member of the wide range of Phonak receivers.

MyLink is compatible with all hearing instruments with a T-coil.



### Online support

To find setup tips and user guides for any combination of Phonak transmitter and receiver with any hearing instrument or cochlear implant, please visit:

[www.phonak.com/FM\\_Configurator](http://www.phonak.com/FM_Configurator)

Please visit [www.eSchoolDesk.com](http://www.eSchoolDesk.com) for more information about FM in schools.





A close-up photograph of the ZoomLink device, a white handheld transmitter with a black control panel featuring three buttons and a small display. The device is set against a bright green background.

# ZoomLink

## D) Large LCD screen

With FM channel, battery status and microphone mode.

## E) Digital multi-frequency channel setting

ZoomLink features an integrated digital multi-frequency synthesizer. Insert a small pen point here to change the FM frequency.

## F) Socket for audio sources

## G) Socket for extra external microphone

## H) Power supply for fast charging

- ZoomLink has a state-of-the-art Li-ion battery
- ZoomLink will work for 10 hours on a full charge
- Charging to 80% in one hour and to 100% in two hours
- Standard or car charger available
- Twin charger available for simultaneous charging of ZoomLink and MyLink

## ZoomLink for everyday life

ZoomLink's easy operation and choice of microphone modes will quickly make wireless communication a part of your client's everyday life. Adults and teenagers, BTE users or users of custom products, can all tune in for a crisp sound!

- Effortless communication in groups at work, in restaurants or at family gatherings
- Easy understanding of lectures
- Hassle-free access to the television, stereo or personal computer

## Tune in for a Crisp Sound with ZoomLink!



## ZoomLink is an easy and flexible FM transmitter for adults and teenagers

- Superior performance in noisy situations
- Optimal reception angle in all listening environments
- Ideal in combination with universal MyLink receiver

## At the Pulse of Innovation

The Phonak Group specializes in the design, development, production and worldwide distribution of technologically advanced wireless and hearing systems. The Phonak Group offers a complete range of digital hearing instruments, along with high tech specialty products and complementary wireless

communication systems. The combination of expertise in hearing technology and a strong cooperation with hearing health care professionals allows Phonak to make a substantial improvement to the quality of life of individuals with hearing impairment and those close to them.