

# CONNEVANS PATIENT SUPPORT AND RESIDENTIAL CARE PACK

The Connevans Patient Support and Residential Care pack is intended for all residential health care situations.


Included in this pack are a variety of products that will enable hospital staff to communicate more effectively with patients who are deaf or hard of hearing. The equipment will also enable patients to use television and radio facilities and participate socially with other patients, visitors or friends.

## SMALL CASE

**Crescendo 20 Plus**  
– with neckloop and headphones for patients with or without hearing aids



The Crescendo can be used for one to one communication or for TV and hi-fi listening. The staff/visitor wears the microphone, the patient wears the headphones or neck loop. The headphones are for use by patients who do not wear a hearing aid. The neckloop is for use by patients with hearing aids which have a 'T' facility.

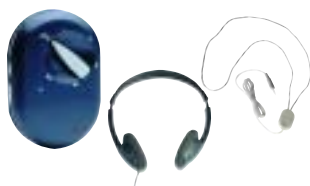

## MEDIUM CASE

**Communicator** – This could be described as an electronic ear trumpet for non-hearing aid wearers



This is a lightweight stetoclip incorporating an amplifier with built in microphone. Designed for use with the elderly, bedridden and infirm and ideal for use where health professionals need to communicate with patients.

The wearer puts the earpieces in their ears with the control unit hanging down. There is a combined ON/OFF switch with Hi/Lo tone adjustment and a volume control.



*The following two items are for personal use by those with hearing aids with a 'T' facility. Both products can be plugged into any equipment which has a headphone socket eg. personal radios, minidisks, CD players, cassettes, talking book machines etc. Most hearing aids have their microphones turned off when turned to 'T' so don't be surprised if a user does not hear you!*

### Personal Inductive Neck Loop

A personal inductive neck loop which can be plugged in any conventional headphone socket. Plug it in, slip the neck loop over the head and switch the hearing aid to 'T'.



### Silent Headphones

These are a pair of headphones with special coils that work with the hearing aid's 'T' facility. Used with two hearing aids they will give a stereo signal, when used with mono equipment only one side will be heard.



## LARGE CASE

**Minicom 6000**  
– a textphone for those who are deaf or who have a speech impairment



### What is a textphone?

A textphone has a keyboard and scrolling character display screen. Instead of speaking into a telephone mouthpiece, you type what you want to say and it appears on the textphone screens of both yours and the person you are calling.

A textphone requires the use of an ordinary telephone line. There are two methods of connection either directly to a BT socket using the lead and doubler supplied or indirectly by placing the telephone handset into the acoustic couplers on the top of the unit.

Making a text phone call is easy, dial the number and wait for a reply. When making a Textphone call the two callers take it in turns to type.

*Full details of textphone use are included in the user instruction guide.*



### Doorknock Beacon

– for patients with or without hearing aids



Compact unit which hangs on the inside of the door and flashes when someone knocks on the other side. The product is activated by vibration rather than sound.



### Telephone inductive coupler – for use with or without hearing aids



This neat product clips over the handset of the phone and can be used either to increase the volume for those with mild hearing loss, or used with a hearing aid switched to the 'T' position.



BPI 36SPK1 Care Pack quick guide



**CONNEVANS LIMITED** 54 Albert Road North, Reigate, Surrey, RH2 9YR  
Switchboard 01737 247571 Text/Minicom 01737 243134 Email mail@connevans.com

# SPARES AND RE-ORDER PART NUMBERS

## SMALL CASE



**Headphones**  
Connevans part no:  
.S20HEAD1



**Crescendo 20 Plus system  
with inductive neck loop**  
Connevans part no: .S20/5

**Replacement Alkaline  
PP3 battery**  
Connevans part no:  
MBNPP3X1



## MEDIUM CASE

**Communicator**  
Connevans part no:  
.ACOM1



**Replacement  
silver oxide 675  
battery**  
Connevans part no:  
\*P009HA\*



**Personal  
inductive  
neck loop**  
Connevans part  
no: 95SH



**Silent headphones**  
Connevans part no:  
95SILHS1

## LARGE CASE



**MINICOM 6000**  
Connevans part no:  
40TM6000

**Standard doorknock beacon**  
Connevans part no: .SDB



**Replacement Alkaline PP3 battery**  
Connevans part no: MBNPP3X1



**Telephone inductive  
coupler/listening aid**  
Connevans part no: 94PL10



**Replacement pack of 4 Alkaline  
AAA batteries**  
Connevans part no: MBNAAAX4

