



Solutions to improve the quality of life

**This product may be purchased from Connevens Limited secure online store
at www.DeafEquipment.co.uk**



DeafEquipment.co.uk
Solutions to improve the quality of life





T-LOUD!

TELEPHONE HANDSET AMPLIFIER WITH AUTO SHUT-OFF

Increases listening volume by 50%

INSTALLING THE T-LOUD

1. Open the battery cover by sliding the cover. (Fig A)
2. Insert standard 9V battery. and place it into the compartment. (Fig B)
3. When replacing the battery disconnect the *T-LOUD* from the phone.
4. Unplug the handset cord from the phone (normally in the base). A screwdriver may be necessary for this to push the tab down on to the connector.
5. Plug the short cord attached to the *T-LOUD* into the base of the phone where the handset was originally. (Fig C)
6. Plug the handset into the *T-LOUD*
7. VELCRO™ strips are provided - the *T-LOUD* can be attached to the phone or the desk for security.

That's it - you're connected!

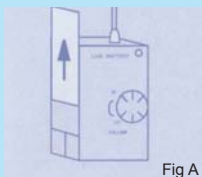


Fig A



Fig B



Fig C

USING THE T-LOUD

1. The *T-LOUD* will switch on and off automatically every time you lift or replace the handset of the phone.
2. Use the phone normally and turn the rotary control to increase the volume to a comfortable level.
3. When the battery need replacing the red light (LED) will come on for a few seconds when the handset is lifted. The *T-LOUD* will continue to operate for a few hours but it is advisable to get a new alkaline battery (type:PP3) as soon as possible.

IF THERE IS A PROBLEM

1. Check the battery
2. Remove the *T-LOUD* from the phone and connect the handset normally by plugging into the base of the phone. If the phone works OK it may be that the *T-LOUD* has a fault. If the problem persists it is likely that there is a problem with your phone. For service, please return the *T-LOUD* to the place you purchased it from. Submit a proof of purchase if this is a warranty claim. The *T-LOUD* is guaranteed for a period of 1 year from date of purchase.

OTHER PRODUCTS FROM SENSORCOM

The *T-LOUD* is one of a range of products to help hearing impaired users to use telephones. We also have the *T-Link* - a low cost accessory that allows hearing aid users the chance to use mobile phones (cellphones) without interference or buzzing.