**This product may be purchased from Connevens Limited secure online store  
at [www.DeafEquipment.co.uk](http://www.DeafEquipment.co.uk)**

**DeafEquipment.co.uk**

Solutions to improve the quality of life

## KEY FUNCTIONS:

	Selects Vibrating Alarm		Snooze Alarm
	Disable Alarm		Lock (disables all "Set" buttons)
	Selects Audible Alarm		Set Time
	Stop Alarm + Illuminate Display		Set Alarm

## 1. TO SET TIME

Slide SET switch to "⌚". Press HOUR/MINUTE to actual time. If your **shake awake** has an AM/PM display, "PM" indicator will appear when afternoon hours are in effect. After setting slide switch to "🔒" (LOCK).

## 2. TO SET ALARM

Slide SET switch to "🔔" (AL) appears on display. Press HOUR/MINUTE continuously until desired alarm time appears. After setting slide switch to "🔒" (LOCK).

## 3. TO ACTIVATE ALARM

Choose VIBRATING "🔊" or AUDIBLE "🔊" alarm. "🔊" will appear on display. When alarm activates slide ALARM switch to "🔔" to disable alarm or press "🔔" to stop alarm. ("🔊" will still appear and alarm will repeat every 24 hours.)

## 4. TO ACTIVATE ALARM WITH SNOOZE FUNCTION (ZZZ)

Slide SET switch to "🔔". Press HOUR/MINUTE continuously until desired alarm time appears. Slide SET switch to "ZZZ" position. (ZZ) will appear on display. Choose VIBRATING "🔊" or AUDIBLE alarm "🔊". When SNOOZE alarm activates press "🔔" to stop. "ZZ" will flash continuously and alarm will repeat every 8 minutes. To disable SNOOZE alarm return SET switch to "🔒" position. (ZZ) will disappear from display.

(Note: To have SNOOZE alarm repeat every 8 minutes you must disable alarm within 1 minute from start. Flashing (ZZ) on the display indicates SNOOZE alarm is still activated.)

## REMINDER:

**shake awake** requires 1 AAA (included) and 2 x AA (not included) batteries. Replace AA batteries when vibrating motor becomes weak. When display becomes dim replace AAA battery.

NOTE: Jack on the right side is an auxiliary output jack for an optional **shake awake** strobe light.

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