**This product may be purchased from Connevens Limited secure online store  
at [www.DeafEquipment.co.uk](http://www.DeafEquipment.co.uk)**




**DeafEquipment.co.uk**

Solutions to improve the quality of life

# HearEasy 310ra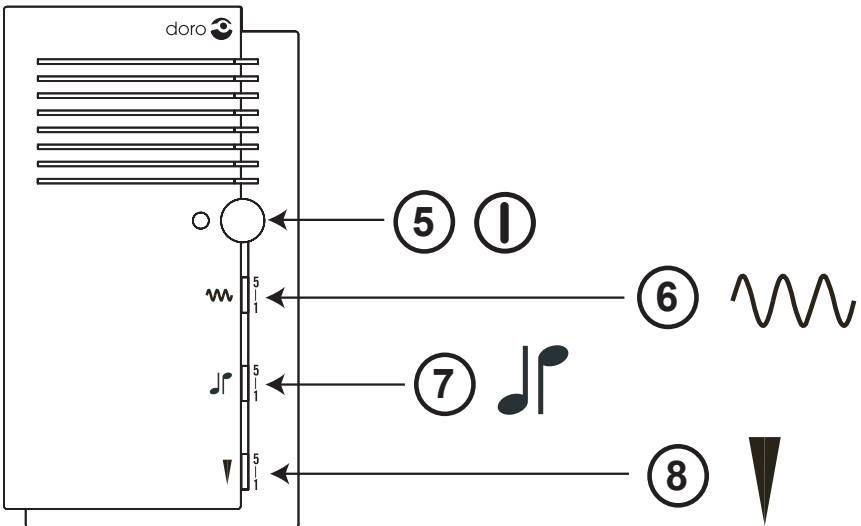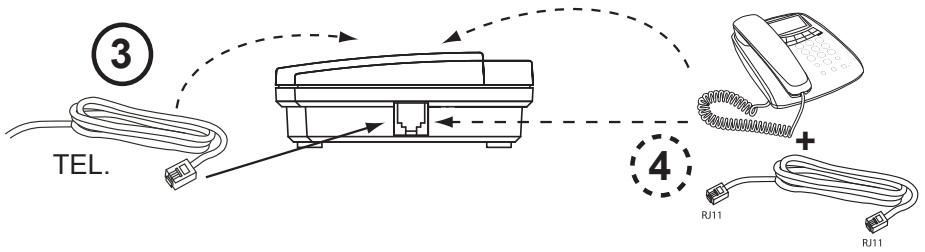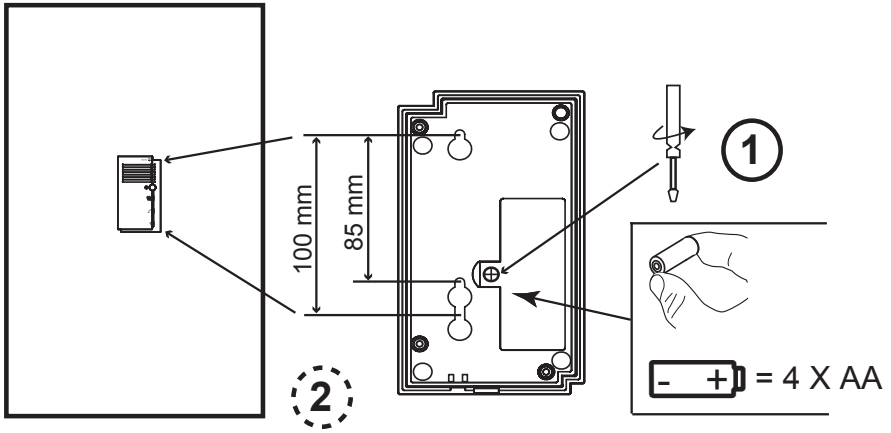

# English

Circled numbers refer to the corresponding illustration.

- ① Insert four AA batteries into the battery compartment located on the underside of the unit. Ensure that the correct battery polarity is observed as shown by the label in the battery compartment.
- ② **Optional:** The unit can be wall mounted. Secure two screws, vertically positioned at 83 mm or 100mm apart.
- ③ Connect the telephone cord to the upper or lower socket () on the unit and then to the telephone network socket. It makes no difference which socket is used.
- ④ **Optional:** You may connect a telephone to the free socket on the unit using the included **RJ11-RJ11** cable. Please note that some phones might be wired differently and will not work with the supplied cable.
- ⑤ Switch the unit ON/OFF.  
Please note that the green LED will not indicate when the unit is switched ON. It will be lit when there is a ring signal present.
- ⑥ Select ringer tone (1-5).
- ⑦ Select ringer type (1-5).
- ⑧ Select ringer volume (1-5).

## Troubleshooting

- Change the batteries if necessary.
- Make sure the unit is switched ON.
- Check that the telephone cord is undamaged and properly plugged in. Disconnect any additional equipment that may be connected. If the problem is resolved, the fault is with the other equipment.

## UK

If you cannot resolve the fault using the faultfinder section, technical support is available by email: [tech@doro-uk.com](mailto:tech@doro-uk.com) -**premium rate telephone number: 0905 895 0854**, calls cost **50 pence per minute** (prices correct at the time of publication).

You can also contact us in writing (regarding any Spares or Technical query):

Customer Services

Doro UK Ltd

1 High Street

Chalfont St Peter

Buckinghamshire

SL9 9QE.

Further contact details are available on our website: [www.doro-uk.com](http://www.doro-uk.com).

# Declaration of conformity

## Declaration of conformity (GB)

Doro declares that the apparatus "RingPlus" complies with the essential requirements and other relevant positions of Directive 1999/5/EC (R&TTE), 2002/95/EC (RoHS). A copy of the Declaration of Conformity can be found at [www.doro.com/dofc](http://www.doro.com/dofc)

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