


PGLSR1

Connections between the Micro and the Loop amplifier

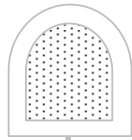


Boundary Layer Microphone in 'MIC 2' socket - Optional.
NB. Sound from this mic' will go through the LOOP but will NOT be broadcast from the MICRO.


You may use the MICRO whilst it is plugged into the mains and charging. However, you might notice a slight 'hum' on the loop system when using the MICRO on mains power. If hearing aid users find this too intrusive use the MICRO on battery power. not mains power.

The MICRO should provide up to 8 Hours talk time when fully charged.


Soundranger Micro Receiver/Amplifier

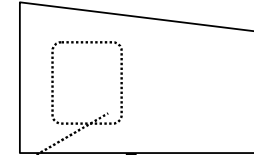

ILD60 Loop Amp. Viewed from the Back.

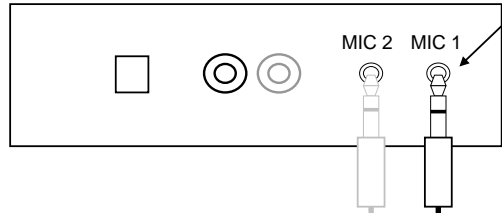

Mono jack
3.5mm

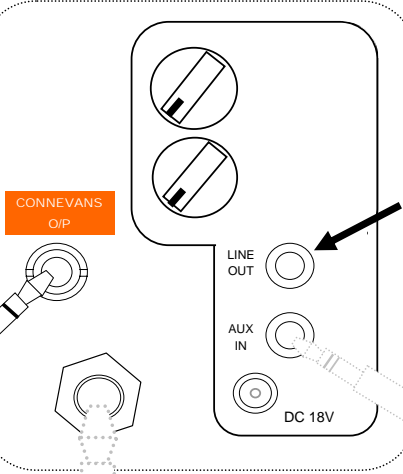

DO NOT USE
(inserting a jack in this socket will mute the loudspeaker !)

6.3mm jack socket - for optional wired microphone.



Optional Connecting lead
Connevens Part no.
A121B (1.20m) or
*A121BA (5.00m) for Auxillary
Input connection.
Eg. CD/Cassette player.

CONNEVANS LIMITED
54 Albert Road North,
Reigate, Surrey RH2 9YR
Tel 01737 247571
Fax 01737 223475
www.connevens.com

Connecting lead - Connevens Part no. *A117* (1.20m)
plus (optional) *A121A* 3m extension