

**This product may be purchased from Connevens Limited secure online store
at www.DeafEquipment.co.uk**



DeafEquipment.co.uk

Solutions to improve the quality of life



How To Put Ear Gear On a BTE Hearing Aid



1. Remove the earmold together with its tubing from the earhook, leaving the earhook attached to the behind-the-ear (BTE) aid.



2-3. Insert the BTE earhook through the end of the Ear Gear with the larger opening and slip Ear Gear over the body of the BTE until the earhook exits through the smaller end of the Ear Gear sleeve.



4. Reattach the earmold and tubing to the BTE earhook.

If the Ear Gear Original is a corded model, attach the clip to clothing