

Tutor courtesy card - EasyLink+ & MLxi



The EasyLink+ and MLxi radio system can be a great help to a hearing aid user - a little of your time to help understand what does what will be much appreciated.



The tutor wears the EasyLink+ radio microphone to pick up their voice, the student wears an MLxi radio receiver to feed sound directly into their hearing aid. The result is a greatly improved and relaxed listening experience for the hearing aid user.

There is no need to be self conscious in wearing a radio microphone - all the best TV stars do so.


The lavalier cord is also the aerial, but please note - if it is long enough to go over your head, it will sit too low to give good sound quality to a hearing aid user. Adjust by squeezing front pad and pulling the free end of the cord.

6 information points


1 The student should give you the EasyLink+ transmitter turned on and ready to use - if you can see the display then the unit is on.



4 During the lesson the EasyLink+ will *automatically* adjust to louder voice levels so you need not change the way you normally talk.

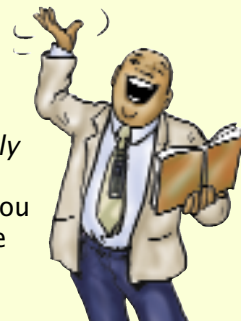

2 Microphone should be positioned about the length of a biro from the mouth.

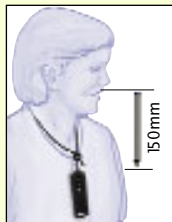

5 During the lesson try not to draw attention to the hearing aid user but do help them by always repeating questions from the floor.

3 Avoid wearing 'noisy' clinking jewellery.



6 Be aware that you have a direct sound link to MLxi users in the room and turning off the transmitter when you are talking to others.

Tutor courtesy card *continued*

EasyLink+ & MLxi



The battery status is shown in the display and a flat battery is shown as an empty battery icon.



The EasyLink+ needs daily charging.

Connection facilities into AV equipment are available using the lead supplied, please contact us for more information.



After the lesson, if you are not returning the transmitter to the user, remember to turn it off before going to the staff room or toilet! The EasyLink+ has a transmitting range of several classrooms.



Thank you for your time.

PS - general interest suggestion: ask your audiology contact to demonstrate what a hearing aid sounds like and how the Phonak MLxi system helps.

BEAPMLICC V2.0



CONNEVANS LIMITED

54 Albert Road North, Reigate, Surrey RH2 9YR
Helpline 01737 247571

Information: www.connevens.com

Online shopping: www.DeafEquipment.co.uk

info@connevens.com