

fmGenie radio system and neck loop for visiting hearing aid users



This loan equipment will enable you to hear more clearly and help you to enjoy your visit.



The speaker wears the radio microphone transmitter to pick up their voice.

You wear the radio receiver to feed the sound directly to your hearing aid via the neck loop or to headphones.

The receiver and its controls

- Turn on the fmGenie receiver.
- Set your hearing aids to the 'T' setting.
- Adjust the **volume** (marked 'Tx vol') of the receiver until you can hear clearly at a comfortable level through your hearing aids.



Congratulations – you should now be able to relax and just enjoy listening!

INFORMATION POINTS

- 1 Check the number of battery bars showing on the right of the display. Two bars or more indicate enough battery to last the day.



- 2 The speaker wears the radio microphone transmitter on a neck harness so that the microphone of the transmitter is positioned about the length of the biro from the mouth. If it is too far from the speaker's mouth, you will not get such a clear signal.

- 3 The radio receiver can either be clipped to your belt in the pouch provided, placed safely in a pocket or held in your hand.



- 4 Once you have adjusted the Tx volume setting of the receiver to suit you personally, the fmGenie will automatically adjust to louder (or quieter) voice levels so you should not need to readjust the volume of the receiver or your hearing aids. Your hearing aids should remain on the "T" setting as long as you wish to listen to the speaker wearing the transmitter.
5. In some areas, the neck loop system may suffer interference from other electrical equipment such as lighting, computers, etc. If you should experience such interference, simply move to a position further away from the source of interference.

Connevens have two websites that you may find of interest



www.connevens.com – this website has lots of information plus our entire catalogue online

www.DeafEquipment.co.uk – the shopping website for all Connevens products plus online special offers



CONNEVANS LIMITED
Telephone: 01 737 247571

54 Albert Road North, Reigate, Surrey RH2 9YR
Fax: 01737 223475 Minicom: 01737243134

fmGenie radio system and neck loop for visiting hearing aid users



This loan equipment will enable you to hear more clearly and help you to enjoy your visit.



The speaker wears the radio microphone transmitter to pick up their voice.



Neckloop for use with hearing aid on 'T' setting

Headphones for non-hearing aid users

You wear the radio receiver to feed the sound directly to your hearing aid via the neck loop or to headphones.

The receiver and its controls

- Turn on the fmGenie receiver.
- Set your hearing aids to the 'T' setting.
- Adjust the **volume** (marked 'Tx vol') of the receiver until you can hear clearly at a comfortable level through your hearing aids.



Neck loop plugs in here

Volume control



Congratulations – you should now be able to relax and just enjoy listening!


INFORMATION POINTS

- 1 Check the number of battery bars showing on the right of the display. Two bars or more indicate enough battery to last the day.



- 2 The speaker wears the radio microphone transmitter on a neck harness so that the microphone of the transmitter is positioned about the length of the biro from the mouth. If it is too far from the speaker's mouth, you will not get such a clear signal.

- 3 The radio receiver can either be clipped to your belt in the pouch provided, placed safely in a pocket or held in your hand.

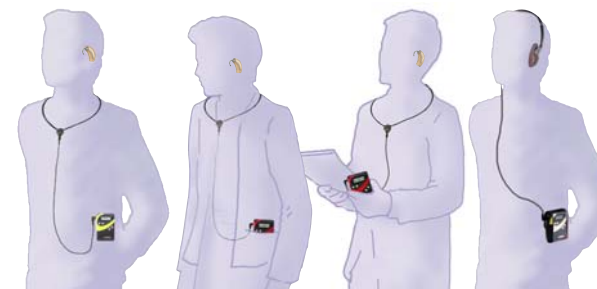

- 4 Once you have adjusted the Tx volume setting of the receiver to suit you personally, the fmGenie will automatically adjust to louder (or quieter) voice levels so you should not need to readjust the volume of the receiver or your hearing aids. Your hearing aids should remain on the "T" setting as long as you wish to listen to the speaker wearing the transmitter.
5. In some areas, the neck loop system may suffer interference from other electrical equipment such as lighting, computers, etc. If you should experience such interference, simply move to a position further away from the source of interference.

Connevens have two websites that you may find of interest



www.connevens.com – this website has lots of information plus our entire catalogue online

www.DeafEquipment.co.uk – the shopping website for all Connevens products plus online special offers



CONNEVANS LIMITED
Telephone: 01 737 247571

54 Albert Road North, Reigate, Surrey RH2 9YR
Fax: 01737 223475 Minicom: 01737243134