

## Tutor courtesy card – fmGenie Tx & Phonak MLx/MLxS


The fmGenie Tx and Phonak MLx/MLxS radio system can be a great help to a hearing aid user – a little of your time to help understand what does what will be much appreciated.



fmGenie Tx



Phonak MLx/MLxS


The tutor wears the  radio microphone to pick up their voice, the student wears a radio receiver to feed sound directly into their hearing aid. The result is a greatly improved and relaxed listening experience for the hearing aid user.

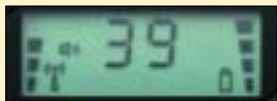
There is no need to be self conscious in wearing a radio microphone – all the best TV stars do so.

It is important that you are comfortable wearing the fmGenie. Please ask for a different harness if you wish.

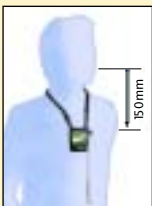


### 6 information points

- 1 The student should give you the radio microphone  turned on and ready to use – if you can see the display then the unit is on.



- 2 Microphone should be positioned about the length of a biro from the mouth.



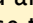
- 3 Avoid wearing 'noisy' clinking jewellery.



- 4 During the lesson the fmGenie will *automatically* adjust to louder voice levels so you need not change the way you normally talk.



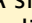
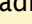
- 5 During the lesson try not to draw attention to the radio aid user but do help them by always repeating questions from the floor.


- 6 Be aware that you have a direct sound link to radio aid users in the room and mute the microphone when you are talking to others – always use the  button muting facility – do not use the ON/OFF power switch during lectures.

## Tutor courtesy card *continued*

## fmGenie Tx & Phonak MLx/MLxS

### Muting the microphone

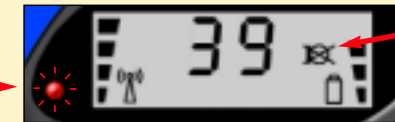
A short press of the  button will mute the radio microphone. Another press of the  button returns to normal.

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
If the feature is active the alert indicator will flash (as a reminder that users cannot hear you) and the microphone mute  symbol will come on.



Star function button for microphone muting

Alert light flashes while microphone is muted



The microphone is muted until you press the  button again.

If the alert light unexpectedly starts flashing there will be a technical problem to be sorted out – probably a broken lead or flat battery, ask your support contact for help.



Connection facilities into AV equipment are available, please discuss your requirements with your audiology support contact.



After the lesson, if you are not returning the transmitter to the user, remember to turn it off before going to the staff room or toilet!

The fmGenie has a transmitting range of several classrooms.



*Thank you for your time.*

PS – general interest suggestion: ask your audiology contact to demonstrate what a hearing aid sounds like and how the fmGenie and Phonak MLx/MLxS system helps.

BFMGMLXCC V1.1



### CONNEVANS LIMITED

54 Albert Road North, Reigate, Surrey RH2 9YR

Helpline 01737 247571

Information: [www.connevens.com](http://www.connevens.com)

Online shopping: [www.DeafEquipment.co.uk](http://www.DeafEquipment.co.uk)

[info@connevens.com](mailto:info@connevens.com)