

**This product may be purchased from Connevens Limited secure online store
at www.DeafEquipment.co.uk**



DeafEquipment.co.uk
Solutions to improve the quality of life



DOOR BEACON DOOR KNOCK RECEIVER

MODEL DBR433

OPERATING INSTRUCTIONS:

Open battery compartment cover, install 3 AA alkaline batteries, and close battery compartment cover.

Placement of unit is flexible and should be determined to achieve optimum viewing of the flashing light. To attach the included "Over the Door" Wire Hanging bracket, Insert from bottom of unit and slide upward until bracket is secure in slots at bottom of unit. Unit may also be placed on a shelf etc. or positioned on a door or wall by using the Velcro included.

Slide switch to ON position to activate. Unit should be in OFF position when not in use.

Easy to set Selectable Frequencies are located inside battery compartment. Set the numbered dip switches to match the transmitter dip switches being used. Different settings can be used to allow for use of more than one transmitter/receiver system without interference.



To replace the light bulb: (290 ma, 4.8 volt required; one replacement included)

- 1) Remove 4 screws in back of cabinet.
- 2) Remove Battery Compartment Cover and Cabinet back.
- 3) Remove screw in middle of circuit board
- 4) Lift printed circuit board.
- 5) Unscrew bulb (small pliers may be required)
- 6) Install replacement 290ma, 4.8Volt Light Bulb.

9 Volt DC Jack allows for connection of an AC Adapter to use the **DOOR BEACON DOOR KNOCK RECEIVER** with AC Electrical power rather than Battery.

ONE YEAR LIMITED WARRANTY:

DOOR BEACON DOOR KNOCK RECEIVER Model DBR433 is warranted to be free from defects in material or workmanship for one year from the date of purchase. This warranty does not include light bulb. This warranty does not apply to any unit which has been used for other than its intended use, or has been subjected to misuse, negligence or accident, or has been opened or serviced by anyone other than an authorized GLOBAL ASSISTIVE DEVICES, INC. Service Center. GLOBAL ASSISTIVE DEVICES, INC. may select to replace the entire unit rather than repair it. Before sending unit for service, check to ensure that battery and/or light bulb do not need replacing. For service during the warranty period, return the unit, with proof of purchase date, in a secure package, insured and with shipping charges prepaid. No C.O.D. packages will be accepted. Include your return street address and a note describing the service required. Enclose \$8.50 (check, money order, Visa, Master Card, Discover or American Express) for domestic shipping and handling charges. Other countries please contact us for pricing.

GLOBAL ASSISTIVE DEVICES, INC.

Fort Lauderdale FL USA

Made in China