

**This product may be purchased from Connevens Limited secure online store
at www.DeafEquipment.co.uk**



DeafEquipment.co.uk

Solutions to improve the quality of life



Instructions for using an Olympus WS-210S with an fmGenie Radio Aid BPIFMG920A

The fmGenie can be used with the Digital Recorder to record a lecture or meeting with high quality sound input. The WS-210S can give up to 69 hours 40 minutes of continuous recording time when it is set to SP Mode.

Setting the SP Mode on the WS-210S

1. Turn the unit on by sliding the "HOLD" switch on the right hand side of the unit downwards.
2. The first time you turn the unit on you will need to set the time and date. To do this, follow the instructions on page 6 of the instructions that accompanied the WS-210S.
3. Press and hold the DISP. MENU button for one second or longer while the recorder is stopped.
4. The display will say 'MENU' briefly and then 'MODE' appears flashing in the bottom of the display then press the OK button to select it.
5. Press FORWARD or REWIND button repeatedly until 'SP' appears in the display then press the OK button to select it.
6. Press the STOP button to return the normal standby screen.
7. If you have selected the SP mode this will be indicated by 'SP' next to the microphone symbol.

The microphone sensitivity needs to be set to low ("dictation" mode)

1. Press and hold the DISP. MENU button for one second or longer while the recorder is stopped.
2. Press the FORWARD or REWIND button repeatedly until 'MIC SENSE' appears in the bottom of the display then press the OK button to select it.
3. Press FF or REW button until 'DICTATION' appears in the display then press the OK button to select it.
4. Press the STOP button to return the normal standby screen.

The WS-210S can only be used with the fmGenie Receiver when it on the low setting for direct audio input into the hearing aid(s).

Connection to the WS-210S is as follows

1. Plug the audio input lead for the hearing aids into one of the sockets of the 3.5mm stereo socket doubler .
2. Plug the 3.5mm stereo plug to plug lead into the other socket.
3. Plug the other end of the plug to plug lead into the 'MIC' socket of the top of the WS-210S
4. Plug the 3.5mm stereo socket doubler into the output socket of the fmGenie.

To playback a recording directly from the WS-200S

1. Plug the blue plug direct input 'V' lead into the socket of the WS-210S marked 'EAR' on top of the WS-210S. Plug the other end of the 'V' lead directly into the hearing aids.
2. The normal black plug fmGenie direct input 'V' lead is not suitable and is likely to overload the input of the hearing aids giving poor quality sound.

The files stored on the WS-210S can be downloaded to a computer using the built in USB interface by following the instructions on page 10 of the instruction sheet that accompanies the unit.