

**This product may be purchased from Connevens Limited secure online store
at www.DeafEquipment.co.uk**



DeafEquipment.co.uk

Solutions to improve the quality of life



M o t i v a t o r



Wake up without disturbing others

Connect the 2.5 mm plug on M O T I V A T O R Model MV45 Bedshaker to any Alarm Clock or Signaling Device with a 12 Volt output jack such as the ILY Alarm Clock.

Place the M O T I V A T O R under the pillow or mattress.

M O T I V A T O R vibrates when Alarm Clock or Signaling/Alerting Device is activated. The built in Automatic Thermal Shutoff with Automatic reset ensures Safe Operation. In the event that your signaling device or alarm is inadvertently left on, this thermal protector will stop the vibrations of the M O T I V A T O R and then automatically reset when the unit is safe to start again.

ONE YEAR LIMITED WARRANTY:

M O T I V A T O R Model MV45 is warranted to be free from defects in material or workmanship for one year from the date of purchase. This warranty does not apply to any unit which has been used for other than its intended use, or has been subjected to misuse, negligence or accident, or has been opened or serviced by anyone other than an authorized GLOBAL ASSISTIVE DEVICES, INC. Service Center. GLOBAL ASSISTIVE DEVICES, INC. may select to replace the entire unit rather than repair it. Before sending unit for service, check to ensure that it is getting the required 12 volts for operation. For service during the warranty period, return the unit, with proof of purchase date, in a secure package, insured and with shipping charges prepaid. No C.O.D. packages will be accepted. Include your return street address and a note describing the service required. Enclose \$7.50 (check, money order, Visa, Master Card or American Express) for domestic shipping and handling charges. Other countries please contact us for pricing.

GLOBAL ASSISTIVE DEVICES, INC.
Fort Lauderdale FL USA