

**This product may be purchased from Connevens Limited secure online store
at www.DeafEquipment.co.uk**



DeafEquipment.co.uk

Solutions to improve the quality of life




PRISM™

STROBE LIGHT MODEL SL12

The **PRISM** Strobe Light is designed for use with alerting devices with a 12 Volt DC output jack. A power source cable is included to connect the **PRISM** to the alarm clock or alerting device. When connected, the **PRISM** will flash when the alarm clock or device is activated.

Power Source Cable:

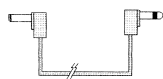

Insert the barrel plug into the **PRISM** and plug the 3.5mm plug into the clock or device.

Place the **PRISM** for optimum viewing. It may be placed on the night table near the alarm clock or mounted using the attached bracket and a screw.

Open the compartment at the bottom of the unit for access to the Dry Contact input jack. This is designed for a Hardwired application of other alerting devices.

Before sending for Service:

If your **PRISM** does not operate, check that the **PRISM** is receiving voltage from the alarm clock or device that it is connected to.

ONE YEAR LIMITED WARRANTY:

Your **PRISM** is warranted to be free from defects in material or workmanship for one year from the date of purchase.

This warranty does not apply to the strobe tube or to any unit that has been subject to misuse, negligence or accident. At its option, Global Assistive Devices, Inc. may replace the entire unit rather than repair it.

For Service, return the device with dated proof of purchase, in a secure package, insured and shipping charges prepaid. Please include a note describing exactly what is wrong with the unit. Include \$7.50 (check, money order, Visa, Master Card or American Express) for return Shipping within the U.S.A. Send unit to:



GLOBAL ASSISTIVE DEVICES, INC

4950 North Dixie Highway, Suite "A"
Fort Lauderdale FL USA 33334-3947