

**This product may be purchased from Connevens Limited secure online store
at www.DeafEquipment.co.uk**



DeafEquipment.co.uk

Solutions to improve the quality of life



ProGuard

HEARING PROTECTION

The perfect solution for anyone concerned about hearing protection

Customised Personal Earplugs

Congratulations on your purchase of ProGuard® Custom Attenuating Earplugs

ProGuard Custom Sound Attenuating Earplugs give your ears the protection needed to safeguard your hearing from excessive noise that can lead to tinnitus, deafness and other noise related symptoms. Fitted with calibrated acoustic filters, your ProGuards will reduce the sound reaching your eardrum without destroying the acoustics and dynamics of the music; think of it as a volume control for your ear. ProGuards are CE marked and comply fully with IEC regulations.

ProGuard earplugs are really comfortable and suitable for long periods of use; they fit discreetly into the ear canal and are made to your exact specification, completely protecting your hearing health in high noise entertainment venues. Manufactured from non-allergenic, soft medical grade silicon they only require the occasional washing in warm soapy water and drying thoroughly before re-using to give years of prolonged use.

Recommended by us, ProGuards can be supplied with a choice of two acoustic filters. Please choose which one you feel applies to the noise environment you expect to spend most time in. Should you want to change the filters afterwards, we will do it free of charge, it will cost you only the postage. Go to www.proguarduk.co.uk to download a returns form.

Filter Colour	SNR	Protection Level	Best for
Green	23	Medium	Regular Musicians/Clubbers
Red	25	High	DJs/Very Loud Bands/Drummers

If you are not delighted with your earplugs then contact us through www.proguarduk.co.uk and we will be happy to talk you through other options: we understand that everyone is different!

ProGuard Custom Earplugs will require impressions of your ears. The process is completely painless and will take no longer than 15 minutes. If you want to get a full understanding of the process where it is fully explained, go to:

www.proguarduk.co.uk/content-audiology.asp

Please follow the instructions on the reverse to register with us and organise an appointment with a local audiologist who will be able to carry out this procedure. After the impressions have been taken and returned to us, our earmould laboratory technicians will use your impressions to create your ProGuards, tailored exclusively to your ears, right down to every bump and curve for a perfect acoustic seal and maximum comfort.



What to do next:

1. Please do not destroy the box as this will be used to send the impressions back to us.
2. Once open please register at www.proguarduk.co.uk, or call 0870 901 6070 telling us where you purchased your ProGuards, and give your name, address, email and importantly, a contact telephone number.
3. We will arrange for you to be contacted and have an appointment made at an Audiology Clinic where impressions will be taken.
4. Once impressions have been taken, please place them inside the box provided and make sure that it is well sealed. Sticky tabs and a reply-paid address label are inside the box.
5. The coupon found at the bottom of these instructions should be filled in with your details. Please place the coupon in the box with your ear impressions and post it to us.
6. Once we receive your impressions, we will manufacture the Custom Plugs to your specification and return them to you within 10-14 days.
7. We will also return your ear impressions, so keep them safe just in case you need to re-order. Your ear shape changes with time and so do your ear canals, so you may also want to consider changing your ProGuards for continued maximum protection.

Contents:

Earplug lubricant, cleaning brush, instructions, voucher, pre-paid postage bag, earplug case.

Lifetime Guarantee:

This product is guaranteed for any problems that arise from manufacture. It does not cover normal wear and tear or loss of the product.

It is also recommended that your ears are checked every two years for any anatomical change which is not covered by this warranty.

Please also see our audiology section on www.proguarduk.co.uk

Serial number:

Earplug Reference Code: PG1998

Make a note of the Earplug Reference Code and quote it if you need to make any enquiries. The contact number is 0870 901 6070.

This is what your earplugs will look like when you receive them.



Marketed by:

Sensorcom Ltd, 4 Thayers Farm Road, Beckenham BR3 4LZ, UK.
Email: info@proguarduk.co.uk Web: www.proguarduk.co.uk

Privacy Policy: Please be aware that the information you provide will be used solely as a means of contacting you regarding your purchase. We shall keep your details **totally confidential** at all times and never ever pass them on to any third party.

(Please print clearly and return this coupon with your ear impressions)



Name

Address

.....

..... Postcode

Email

Telephone

Mobile

Serial number:

