



Solutions to improve the quality of life

**This product may be purchased from Connevens Limited secure online store
at www.DeafEquipment.co.uk**



DeafEquipment.co.uk
Solutions to improve the quality of life



“*VibrasOUND*”™

Extra Loud Alarm Clock
with
Variable Volume,
Variable Tone,
& Auxiliary Jack
for Bedshaker or Light

Model VSC12

The “*VibraSOUND*”™ Digital alarm Clock has Variable Loudness and Variable Tone for the audible alarm. The Auxiliary Jack on “*VibraSOUND*”™ allows you also to wake up to vibration by connecting our “L’IL BEN”™ or “Vibes”™ 12 Volt Bedshaker or wake to a flashing light by connecting our “Prism”™ 12 Volt Strobe Light (accessories are sold separately).

“*VibraSOUND*”™ is custom manufactured by GLOBAL ASSISTIVE DEVICES INC. for people with special needs for wake up. You can select the tone and loudness for the audible alarm that meets your personal requirements or situation. With the auxiliary jack provided, you can add the low voltage “L’ IL BEN”™ or “Vibes”™ bedshaker and place it under your pillow or mattress to awaken by vibration.


The “Prism”™ 12 volt Strobe Light is also an optional accessory that can be plugged into the auxiliary jack and will flash when the alarm is activated. With the selection controls you can choose to have a steady or pulse vibration. You can select to have the vibration or flashing light with or without audible sound. The selection of no audible is ideal when you do not wish to disturb others. The user adjustable 50/60 Hz switch Feature allows for reliable use abroad as well as at home. The green LED display provides easy reading of the time.

BEFORE using your unit, take a few minutes of your time to read the following instructions. The understanding of the

features of your new wake up device will allow you ease of setting and effective use.

IMPORTANT:

A 50/60 Hz SWITCH is located inside the battery compartment of the clock. Check the position of the switch

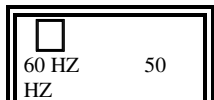

and slide the switch to the position for your country. IN USA AND CANADA THIS SWITCH MUST BE IN THE 60 HZ POSITION.

FOR OTHER COUNTRIES PLEASE CHECK WITH YOUR ELECTRIC COMPANY AND SET THE SWITCH ACCORDINGLY. IF THIS SWITCH IS SET INCORRECTLY, THE CLOCK WILL GAIN OR LOOSE TIME.

OPERATION:

The clock is powered by a 12VAC 1000mA Adapter. (Available in 110 Volt or 220 Volt) At the back of the clock you will find two small jacks (openings). Insert the small single plug of the AC Adapter into the jack marked AC12V INPUT. Connect the other end of the Adapter (the two pronged AC plug) into an AC wall outlet near your bedside. 12:00 will flash on the clock display. When you set the time, the display will stop flashing.

SETTING THE TIME

To set the correct hour of the day, press and hold the TIME button. While you hold down the TIME button, press the HOUR button until it advances to the correct hour of the day. Release buttons when correct hour shows on the clock display.

N.B. The PM INDICATOR LIGHT is a glowing green dot at top left of the clock display. When the clock is reading PM time, the light will be "on". The light will be off when the clock is reading AM time. **BE SURE TO USE THIS GUIDE FOR CORRECT SETTING OF CURRENT TIME** (as well as Alarm Settings).

To set the correct minute of the hour, press and hold the TIME button. While you hold down the TIME button, press the MINUTE button until it advances to the correct minute. Release buttons when correct minute shows on the clock display.

SETTING THE ALARM TIME:

The ALARM SET button is labeled AL. SET.

To set the hour you wish to wake up, press and hold the AL. SET button. While you hold down the AL. SET button, press the HOUR button until it advances to the correct hour. Release the button when desired alarm hour shows on the clock display.

N.B. Use the PM INDICATOR LIGHT to ensure that the setting for alarm is correct. Example: PM INDICATOR LIGHT is OFF for AM time wake up.

To set the correct minutes past the hours you want for wake up, press and hold the AL SET button at the same time as you press the MINUTE button. When the minutes advance to the desired time, release the buttons.

USING THE ALARM:

Set the ALARM selector to the desired mode. (AUDIBLE for audible only, VIBRATE/LIGHT to activate the auxiliary jack for vibration or flashing light; BOTH to give audible & vibration or audible & light.) The ALARM INDICATOR LIGHT(s) on the Display will appear to show setting(s). These light(s) shows that the alarm is set to operate in the mode(s) selected.

To stop the ALARM after it is activated, press the button on top of the clock marked AL OFF. The alarm setting will remain for the same time, for the next day.

To stop the ALARM from being activated at the set time, slide the Alarm Selector switch to the ALARM OFF position. The Alarm Indicator light(s) will turn off.

Whenever you wish to check what time the Alarm is set for, press and hold the AL. SET button. The Clock Display will then show the alarm time that has been set. Check that it is set properly for AM or PM time by using the PM indicator light to confirm time of day you want to be awakened.

If you wish to awaken at the same time each day, there is no need to adjust or reset the alarm time. Just ensure that the

Alarm selector switch is in the desired ON position before going to sleep. (Audible, Vibrate/Light or Both)



VARIABLE LOUDNESS/ VARIABLE TONE

The Volume Dial

allows you to select the alarm loudness you require or prefer. (Alarm selector must be in *Audible* or *Both* position for audible alarm to work)

The Tone Dial allows you to select the frequency/tone of sound that is most effective for your needs.

AUXILIARY JACK FOR 12VDC BEDSHAKER/LIGHT:

Your clock is equipped with an AUXILIARY JACK that allows you to connect a bedshaker (L' IL BEN™, Model SS12 or "Vibes"™ Model VSS12) or our 12 Volt Strobe Light PRISM™, Model SL12 (These accessories are purchased separately). This auxiliary jack is located at the rear of the clock and is labeled DC12V Output. Simply insert the mini plug of the Bedshaker or Light into this jack. To use the bedshaker, place it under your pillow or between your box spring and mattress. You can select either a pulse or constant vibration by adjusting the VIBRATE switch at the top left of the clock. When the alarm is activated, the bedshaker or light will also be activated. When the alarm is turned off, the vibration or light will stop.

Optional Accessories



USING THE SNOOZE FEATURE

After the alarm is activated, press the SNOOZE button. This will stop alarm activity (sound, bedshaker or light). In approximately 9 minutes the alarm(s) will start again automatically. You can press the SNOOZE button to repeat this off/on again cycle as many times as you wish, up to one hour.

BATTERY BACK UP for CLOCK MEMORY

Your clock is equipped with a BATTERY BACK UP system. This system will keep the clock time running and the alarm settings maintained IN THE EVENT OF A POWER FAILURE. The clock display is not illuminated when the clock is running on battery. The correct time will show on the display when the clock is returned to electrical power (providing the battery has adequate power). A prolonged power failure can drain the battery's power. If your clock's display flashes when electrical power is returned, replace the battery and reset the clock time.

Installing/Replacing the Back-up Battery:

The battery compartment is located at the back of the clock. To remove the cover, slide it downward and lift up to remove. Connect a 9 volt battery to the terminals. Alkaline recommended. (Battery not included) Position battery in the compartment. Replace the cover.

CARE AND MAINTENANCE

Your *Vibra* SOUND™ clock has been carefully designed and manufactured to provide you with accuracy and dependability. The following suggestions will help you care for your clock so that you can enjoy the maximum from it.

- . Keep the clock dry. It does get wet, wipe it dry immediately. Liquids can contain minerals that corrode the electronic circuits.
- . Use only a fresh battery of the recommended size and type. Always remove an old or weak battery. It can leak chemicals that can damage electronic circuits.
- . Handle the clock gently and carefully. Dropping it can damage the circuit boards and case and can cause it to work improperly.
- . Use and store the clock only in normal temperature environments. Extreme temperatures can shorten the life of electronic devices and distort or melt plastic parts.
- . Keep the clock away from dust and dirt, which can cause premature wear of parts.
- . Wipe the clock with a dampened cloth occasionally to keep it looking new. Do not use harsh chemicals, cleaning solvents, or strong detergents to clean the clock.

- . Modifying or tampering with your clock' s internal components can cause a malfunction and will invalidate the clock' s warranty.

SERVICE:

The unit should be serviced by qualified service personnel. The user should not attempt to service the unit. All servicing should be referred to:

GLOBAL ASSISTIVE DEVICES, INC.
4950 North Dixie Highway, Suite 121
Fort Lauderdale FL USA 33334-3947

PLEASE NOTE, PRIOR TO SENDING FOR SERVICE:

IF YOUR CLOCK IS GAINING OR LOSING TIME, CHECK THE POSITION OF THE 50/60 HZ SWITCH (located in the battery compartment) TO ENSURE IT IS SET CORRECTLY. IN USA AND CANADA THE 50/60 HZ SWITCH MUST BE IN THE 60 HZ POSITION. FOR OTHER COUNTRIES PLEASE CHECK WITH YOUR ELECTRIC COMPANY.

ONE YEAR LIMITED WARRANTY

* Your *Vibra* SOUND™ is warranted to be free from defects in material or workmanship for One Year from the date of purchase.

* This warranty does not apply to any unit which has been subject to misuse, negligence or accident or has been opened or serviced by anyone other than an authorized Service Depot of Global Assistive Devices, Inc.

* At its option, GLOBAL ASSISTIVE DEVICES, INC. may select to replace the entire unit, rather than repair it.

* During the warranty period, return the clock, with dated proof of purchase, in a secure package, insured and with shipping charges prepaid. C.O.D. packages will not be accepted. Include a note describing the service required and your return street address. No PO Box Numbers Please. Include \$6.50 (check, money order, Visa, Mastercard or American Express) for shipping and handling within the U.S. Outside of U.S., call for quote.

Vibra SOUND TM

Manufactured in China

GLOBAL ASSISTIVE DEVICES, INC.

Fort Lauderdale Florida USA

Model VSC12
