

**This product may be purchased from Connevens Limited secure online store
at www.DeafEquipment.co.uk**



DeafEquipment.co.uk
Solutions to improve the quality of life



Your hearing instrument has an audio input. This enables accessories to be connected. In this way you can understand better in certain hearing situations.

The contacts for the audio input are located under the type plate. Before using the audio input for the first time the covering type plate must be exchanged against the supplied type plate providing openings for the audio contacts. Your hearing instrument specialist will do this for you. The coloured type plates can be used to identify your left (blue) and right (red) hearing instrument.

Your hearing instrument recognizes immediately when an audio shoe is fitted and diverts then automatically to program "4." You can notice this by the four beep tones 🎵🎵🎵🎵.

Fitting the audio shoe to the hearing instrument

Hook the audio shoe in from the front and press it to the rear into the detent on the inner curve of the instrument.

